AC Information necessary for safe use

Document title	AC Information necessary for safe use
Document version	4
Document revision	3
Writer	Oskari Koskimies
Reviewer	Samppa Savonen
Approver	Tuomas Mäkynen
Last modified date	Aug, 26 2025, 18:37
Last modified by	Oskari Koskimies
Template version	1
Status	APPROVED Sep-02-2025, 13:56

1. Table of content

- [1. Table of content](#)
- [2. References](#)
 - [2.1. Internal references](#)
 - [2.2. Citation table](#)
- [3. Information necessary for safe use](#)
- [4. Warnings](#)
- [5. Varoitukset](#)
- [6. Varningar](#)

2. References

2.1. Internal references

ID	Internal References	Space key
----	---------------------	-----------

2.2. Citation table

ID	Cited By	Space key
----	----------	-----------

3. Information necessary for safe use

The warnings relevant to healthcare professionals using the Client System are considered information necessary for safe use. They are provided below in English, Finnish and Swedish languages. The Client System **MUST** display these warnings to all users. If the Client System supports other languages, you **MUST** obtain official translations from Orla DTx for the supported languages before using the Asthma Calculator in production.

4. Warnings

If you have any questions about the warnings in this section, please contact Orla DTx technical support using the contact information below.

WARNING: While every effort has been made to ensure the accuracy of calculation results, the possibility of software error cannot be completely ruled out. Data should not be used as a sole means for determining the patient diagnosis.

WARNING: If the Asthma Calculator reports an error that you are not able to correct, or fails to provide results, please contact Orla DTx technical support using the contact information below.

WARNING: Use of the results provided by the Asthma Calculator is not recommended for diagnosing patients below 12 years of age.

WARNING: PEF EU reference value calculation is not suitable for patients below 15 years of age, or above 85 years of age.

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Orla DTx Oy